

please read this manual carefully  
and understand thoroughly before  
you operate system.

Multifunction slimming machine



**User manual**

## **Content**

Safety notice-----	3
Operation notice-----	4-5
Summarize & theory-----	6-8
Advantage & main function-----	8
Diagram of Accessories-----	9
Operation screen-----	10-14
Detail pictures-----	15
Illustration of operation-----	16-20
Taboos-----	21
Technical parameter & packing list-----	22
Questions & answer-----	23
After-sale service & maintenance-----	24

## **1. Safety Notice**

1.1 Please don't disassemble the equipment or try to do some other operation, which haven't been indicated in our instruction. All the repairing work should be done by our admitted professional personnel

1.2 Please don't assemble or operate this equipment while you are near the water or your hands are wet. Notice that don't pour the liquid on it.

1.3 Please pull out the plug from the socket when you met such conditions as below. And in touch with professional maintenance Station.

- A. The equipment touched liquid
- B. The equipment has abnormal smell, smoking or loud and strong noise.
- C. Cable is broken
- D. The equipment is fallen down and broken

1.4 Please don't put anything on the cables, don't place the equipment at any place where someone can step on the cable.

1.5 Please pull out the power cable after using it for your safety

1.6 Don't put anything into the equipment's jack, otherwise it may cause fire or touch the electricity. If something enters the equipment, please don't pick it out by yourself, contract distributor or our company immediately

1.7 Please don't put the equipment on the anything that can't stand steadily, avoid it turns over and damage.

1.8 Our company has the right to modify the information in this introduction before notify! And keep the explanations right for the above.

## 2. Operation Notice



**Before operation, please read the following content.**

- 2.1 It is essential to make different setting according to specific treatment demand.
- 2.2 The application of The machine must never be painful and cause redness to the skin or leave marks.
- 2.3 Move the treatment heads slowly over the skin of the body or the face. Don't letting them stop in any one place on the skin.
- 2.4 Do not apply the device on children less than 7 years old. The treatment to children over 7 requires medical advice.
- 2.5 Do not use this device if suffering from a fever, inflammation, dermatitis (acne, eczema, herpes), warts, open or recently stitched wounds, serious and progressive illnesses (cancer, AIDS, hepatitis, blood diseases, etc. or blood coagulation problems. Do not apply the device if you have or have had phlebitis, after veins surgery within the last two months, if you have varicose veins or tingling in your limbs, if you have any concerns consult a physician.
- 2.6 Never use the device on a pregnant woman.
- 2.7 The machine could be used on all external and healthy areas of the body except the eyes, throat, ears, glottis and the genital area.
- 2.8 Must not be applied inside the mouth, or on mucous membrane, or on nipples, or moles without medical advice.
- 2.9 Never use the device if you suffer from epilepsy. The device must not be used by any person suffering from mental illness.
- 2.10 In the case of long-term treatment with drugs such as anti-inflammation, anticoagulants, antibiotics or if your skin bruises at the slightest shock, ask advice of your doctor/physician before using.
- 2.11 Never apply to the body parts with jewelry or ring.

2.12 Regular use of The machine enhances well-being; it is better not to stop its use for a prolonged period of time. Not following operating instructions and usage recommendations may bring lesser results than expected. Do not use the device longer than the recommended time.

2.13 If your symptoms have no improvement, or if in any doubt on the use, consult your doctor.

2.14 Effect may vary depending on your age, life habits (level of physical activity, diet, etc.).

2.15 The machine can be used after any plastic surgery only while following the instruction of the surgeon.

2.16 The machine is designed for professional use only, any personal use of this device is prohibited.


2.17 The Machine continuous working time shall not exceed 1 hours. The best way is cut the power off, let the machine have a rest for 20-30mins. After the machine have worked for a hour.

2.18 Never use the strong acid or strong base or alcohol clean the machine. (use physiological saline to clean machine.)

### 3. Summarize & Theory

**Cryolipolysis:** a new, non-invasive way to gently and effectively reduce fat in targeted areas of the body that results in a noticeable, advanced-looking fat reduction in the treated areas. As triglyceride in fats will be converted into solid in particular low temperatures, it uses cooling technology to selectively target fat bulges and eliminate fat cells through a gradual process that does not harm the surrounding tissues, reduce unwanted fat. The contact cooling of the hand piece surface regulates the temperature of the skin and protects fine dermal structures, realizing the fast body-reshape effects while tightening skin!

#### How Cryolipolysis works

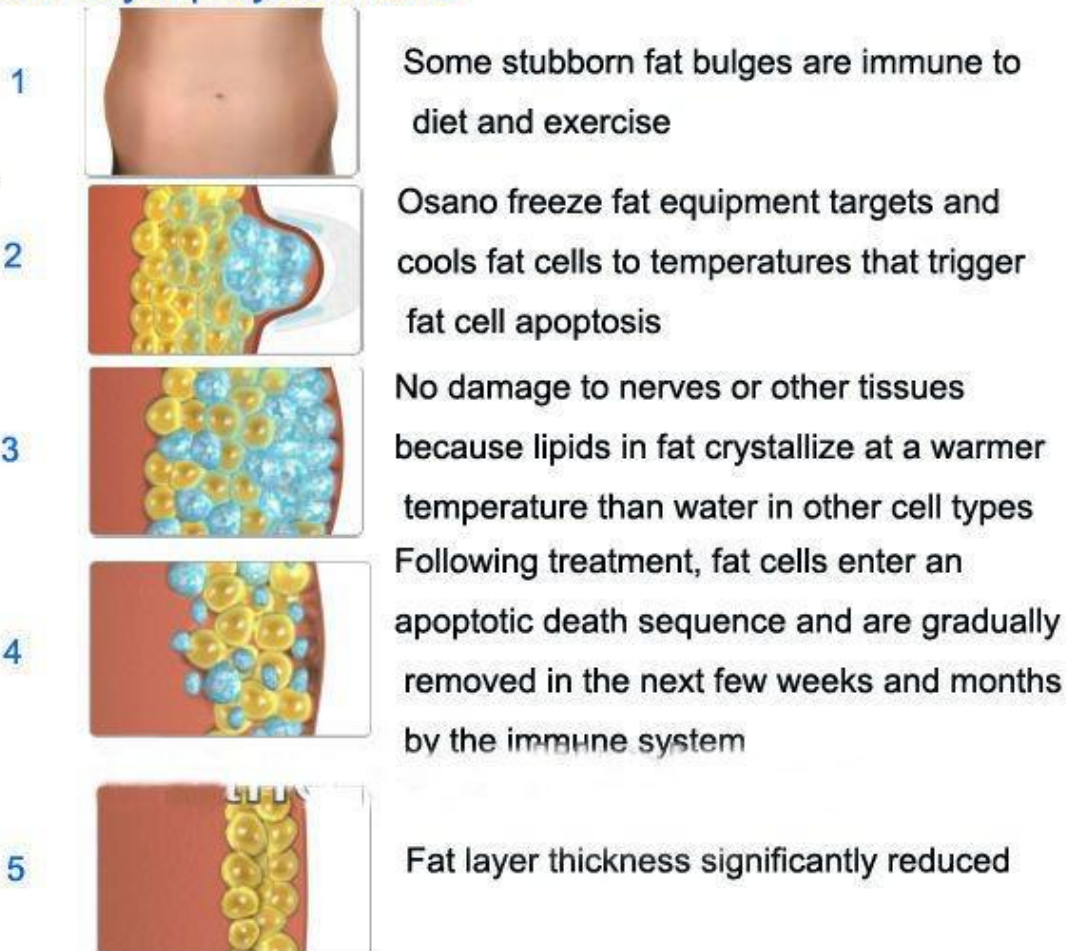

So Cryolipolysis is a great solution for subcutaneous fat layer reduction.

Join Osano to stay away from annoying stubborn fat!(^.^)!

## 40K Cavitation

Strong sound wave explosion fat head with collective strong sound wave head, strong sound wave of 40KHz may be emitted to human body for impacting fat cells fiercely and causing friction motion between fat cells. This may cause effective consumption of calories and moisture in fat cells and reduce the size of fat cells. What's more, sound wave vibration may cause fierce impact of fat cells to make them be exploded instantaneously, reduce the amount of fat cells and thereby achieve the effects of removing fat.



## RF:

Radio frequency is one kind of high frequency electromagnetic wave, it can penetrate the dermis and subcutaneous tissue through the fat layer, take heat into deep skin and then accelerate regeneration and reconstruction of skin deep-tissue. RF can produce energy by impedance from the skin, which can make the fatty layer temperature increase in very short time, and this heat will also act on collagen in the dermis layer. Collagen in the dermis layer will shrink immediately after the temperature rise and proliferation of new collagen, so as to achieve the effect of skin pulling, pigmentation improvement, skin whitening, and wrinkle removal.



## Lipo Laser:

Laser Lipolysis is growing in popularity as a procedure for the tummy, arms, male breasts, hips and outer thighs as well as smaller areas of the body such as redefining the jaw line, neck and chin area. It carries fewer risks as traditional liposuction and is a less invasive surgical procedure requiring fewer incisions and resulting in less scarring. We use laser liposuction after liquid liposuction to tighten the skin.



## 4. Advantages of cryolipolysis:

1. Non-surgical technology
2. Cryolipolysis technology advanced than Lipo surgical technology
3. Popular technology in Europe and American
4. Newest technical to lose weight reduce 26% fat in treatment area
5. New technical is more advanced than RF and ultrasonic.
6. Eliminate body fat part by part where you want to reduce

## 5. Main function:

- 1). Body slimming, Reshape body line
- 2). Cellulite removal
- 3). Localized fat removal
- 4). Lymph drained
- 5). Skin tightening
- 6). Pain relief for relaxation
- 7). Improve blood circulation
- 8). Combine cryolipolysis, cavitation treatment with RF to enhance sliming effect of beauty equipment




## 6. Diagram of Accessories

### 6.1 Cryolipolysis handle:



(2 pcs touch screen cryolipolysis) Size:8\*18cm /5\*16cm

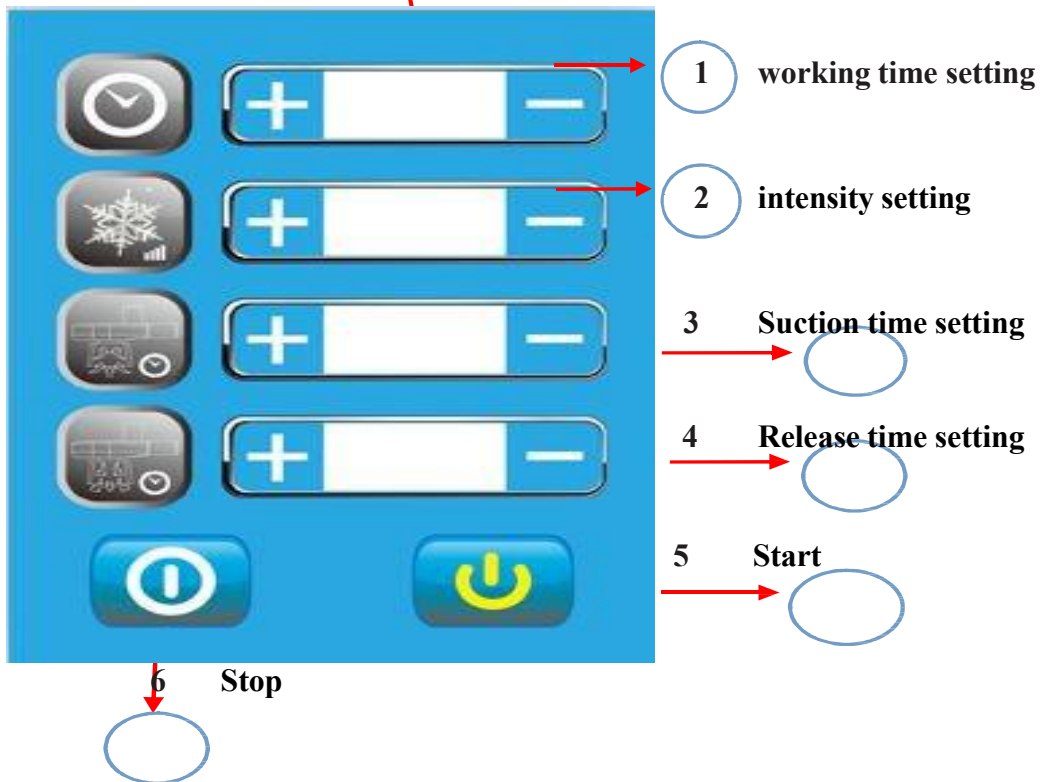

## 7.operation screen



Please touch anywhere to enter to function screen:



## Cryo system:



Cryo lipo 1: big size cryolipolysis handle operation screen




it is cryo lipo 2 and 3 operation screen if you press them

Cryo lipo 2: middle size cryolipolysis handle operation screen

Cryo lipo 3: small size cryolipolysis handle operation screen

Note: cryo lipo 1 and cryo lipo 2 two touch screen cryolipolysis can be working together or working separate according to customer needs.

# CRYO LIPO 2

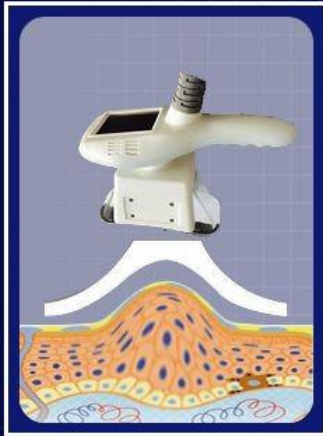

WORK TIME



VACUUM





RELEASE







COOLING





A. Please analyze the fat part status, then according to it, to adjust the Intensity, suction release time and working time.


B. **VACUUM** vacuum setting: click the key of “ ” then click up and down, 0.5s-9.0s, according to the customer's need.

C. **RELEASE** release setting: click the key of “ ” then click up and down, 0s-4.0s, according to the customer's need.

D. **WORK TIME** time setting: click the key of “ ” then click up and down, 1-60min, according to the customer's need.

E. **COOLING** cooling temperature: click the key of “ ” then click up and down, 10 °C to -10 °C, according to the customer's need.

F. Then click the key of “” to start operation

Tips: click the  will be returned back to last screen!

# 40K Cavitation System:

## 40K



**WORK TIME**

↑

min

↓

**ENERGY**

↑

J/cm<sup>2</sup>

↓

Mode 1

Mode 2

Mode 3

Start/stop



## RF



**WORK TIME**

↑

min

↓

**ENERGY**

↑

J/cm<sup>2</sup>

↓

Mode 1

Mode 2

Mode 3

Start/stop



A Please analyzer the fat part status, then according it, to adjust the Intensity and the Wave and Working time.







There are 3 modes: mode1 mode2 mode3

Mode 1 : Suitable for arms, thighs & other parts therapy, or mild obese type.


Mode 2 : Suitable for hip, abdomen & back, or moderate obese type.

Mode 3 : Suitable for arms, abdomen which is moderate obese type.


B. **ENERGY** intensity setting: click the key of “ ” then click up and down, 1-10, according to the customers need.

C. **WORK TIME** time setting: click the key of “ ” then click up and down, 1-30min, according to the customers need.

F. Then click the key of “” to start operation

Tips: click the  will be returned back to last screen!

## Lipo Laser System:

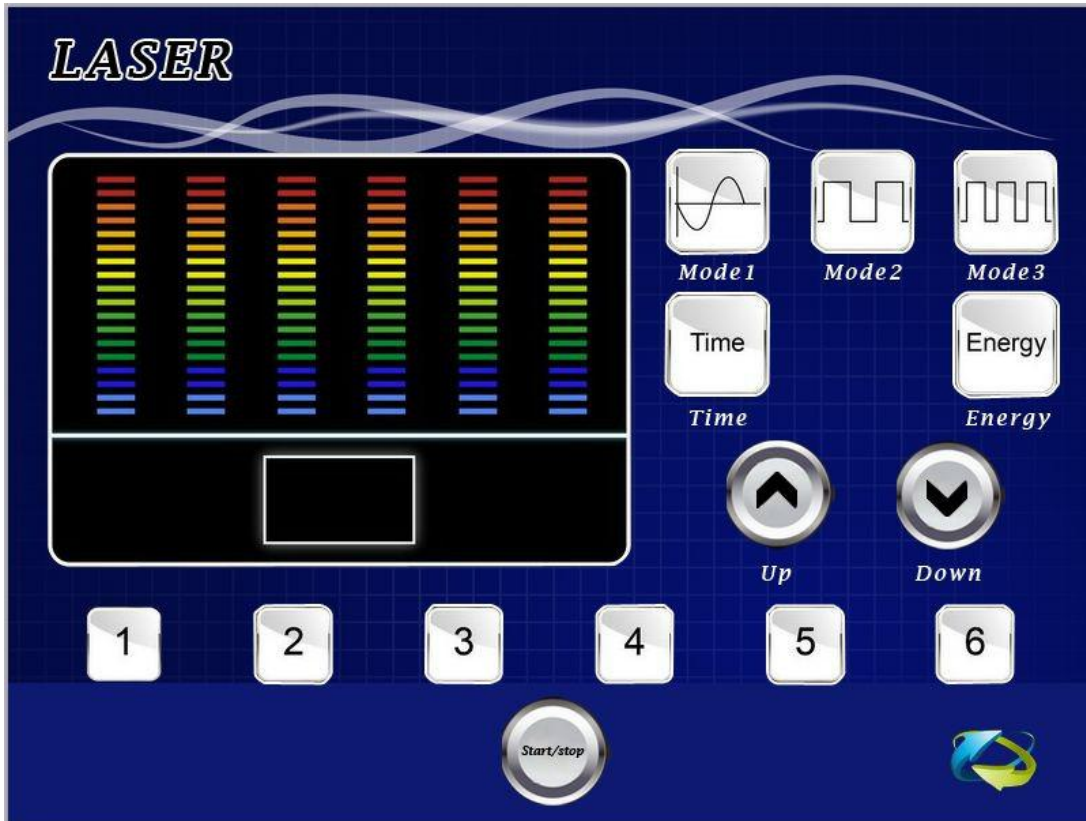

A Please analyzer the fat part status, then according it, to adjust the Intensity and the mode and Working time.





There are 3 modes: mode1 mode2 mode3

Mode 1 : Suitable for arms, thighs & other parts therapy, or mild obese type.

Mode 2 : Suitable for hip, abdomen & back, or moderate obese type.



Mode 3 : Suitable for arms, abdomen which is moderate obese type.



B. **Energy** intensity setting: click the key of “   ” then click up and down, 1-18, according to the customers need.






C. time setting: click the key of ”   ” then click up and down,1-99min,according to the customers need.




they are lipo laser pads,total 6 pads,each one control separate.



F. Then click the key of “  ” to start operation



Tips:click the  will be returned back to last screen!

## 8.Detail pictures:





## 9. Illustration of operation


### THIGH

Treated time: 30 minutes


1. From down to upper, pushing to the groin to dredge the lymph.




2. From down to upper, circling push by anticlockwise, can help decomposing fatness.

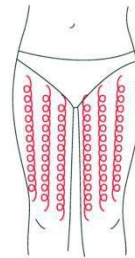

3. Also can push by come-and-go to decompose fatness.

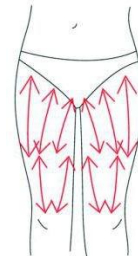


4. Pull from the knee and muscle texture to upper, can improve the curve.

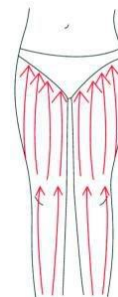


## **BUTTOCKS**

**Treated time:20 to 30 minutes**

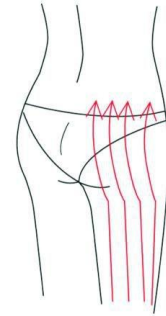

**1. Along the lymph direction, pulling to the waist.**


**2. From upper to down, by anticlockwise gesture, pull come-and-go.**

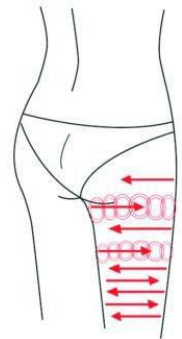

**3. Along the muscle of arm, pull up to lift and tight the muscle.**


## **BACK**

**Treated time: 20 to 30 minutes**

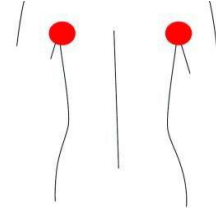


**1. First by come-and-go, pushing the bladder nerve 2 to 3 times.**

**2. Circling by anticlockwise to stimulate the underarm lymph node.**

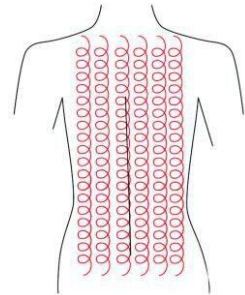

**3. On the back, by anticlockwise circling, can help to decompose the fatness.**

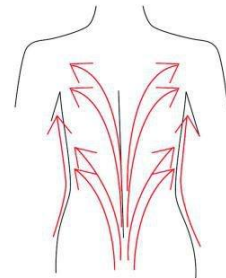

**4. Using lymph drainage gesture, pull the toxin to the lymph node, can help improving the back curve.**

## ARM

Treated time:30 minutes

### BACK SIDE

(5)From the inner elbow to the armpit, doing lymph drainage.



(6)Circling to stimulate the lymph node.



(7)By anticlockwise or come-and-go pushing, decomposing the fatness.



(8)From the elbow to the oexter, doing tightening gesture.


Facade

1. From the elbow to the oexter, doing tightening gesture.


## Abdomen

Treated time:20 to 30 minutes


1. From small to large, by clockwise direction, circling arround the navel, can help peristalsis of the large intestine.

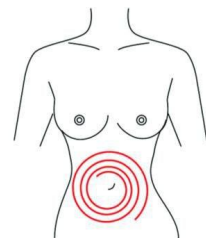


2. From small to large, by anticlockwise direction, circling push down.

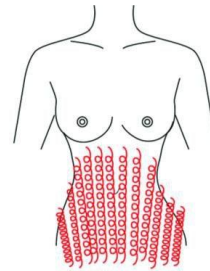

3. From the belly and muscle texture, pull to the groin.

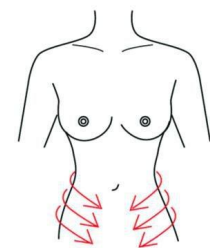

4. Through lymph drainage gesture, taking the toxin to the place of groin.

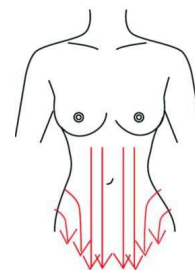



## **10.Taboos(who are not suitable to use this machine?)**

9.1 Pregnant women or women during in menses.

9.2 Epileptic

9.3 Patients with malignancy.

9.4 Patient whose wound after operation has not healed up.

9.5 Acute inflammation or epidemical patients.

9.6 Whom with heart diseases or with heart pacemaker..

9.7 Whom with kidney (gall-stone ) disease.

9.8 Who was embedded metal object or silica gel.

9.9 Who in menses, birth control period, emiction   incontinence period, or accepting the belly operation.

9.10 Whose body always take much inner hot.

9.11 Who has the genetic hypersensitivity.

## 11. ACCESSORIES LIST & TECHNICAL PARAMETER

### Technical Parameter

Voltage	AC110V/220V 50-60Hz
Power	< 800w
Temperature	-10 °C~10°C
Pressure output	0-900Kpa
Handle working size	8cm*18cm/ 6cm*15cm
Screen	15 inch touch screen
Cryolipolysis	-14 °C~15°C
Cavitation	40K/25K
RF	1Mhz
Lipo laser	650nm
Net weight	40KG
Gross weight	55KG
Package size	130cm*64cm*60cm

### Accessories List

Description	Qty.
Main machine	1pcs
Anti freeze mask	10pcs
Cavitation	1pcs
RF	1pcs
Lipo Laser	6pcs
Cryolipolysis handle	2pcs
Water funnel	1pcs
Power cord	1pcs
Fuse	2pcs
Holder	1set
Manual	1books

## **12. QUESTION&ANSWER**

### **1. What is Cryolipolysis Extreme Shaping?**

Cryolipolysis Extreme Shaping provides the most effective solution to improve and control the orange skin and localized fat through a combination of the latest aesthetic technologies, the best results in fewer sessions.

### **2. What are the applications of Cryolipolysis Extreme Shaping?**

The Cryolipolysis Extreme Shaping is effective in controlling and improving the orange skin and localized fat, with skin markings also used as muscle relaxant ....

### **3. On what areas can be applied?**

With Cryolipolysis Extreme Shaping can act anywhere on the body that this orange skin (buttocks, legs, abdomen, arms, etc)

### **4. Does it hurt the application?**

The sessions are not painful Cryolipolysis Extreme Shaping since only non-invasive techniques are used. In addition, the vacuum causes relaxation massage and wellbeing.

### **5. Do I have to do other things to lose the fat?**

Only for bigger results - 1 inches without changes, 5inches with major lifestyle changes, like diet and exercise we have support if you choose to change your life while doing the sessions.

### **6.What do you do to keep the fat off?**

Normal things you need to keep weight off diet and exercise, drinking water, etc.

### **7.Where Does the Fat Go?**

That fat can be burned as energy in the body, or normal metabolic processing just like when you exercise and release fat.

### **8.How Quickly Will I See Results?**

Many clients see measureable results after their first session.

### **9.What Kind of Results are Produced?**

1-5 inches average-the most lost in 9 sessions is 23 inches People who make minor lifestyle changes, are on the higher end of the average, and those who don't, the lower end.

### **13. AFTER-SALE SERVICE AND MAINTENANCE**

The term of maintaining guarantee is main device 1 year and spare part 3 months from the date of purchasing according to the receipt. (Expendable is not included in it.) Others not included in maintaining guarantee or over the time, we also supply to maintain and renew but need charges. Please contact with the dealer if need change of purchasing the parts and easily expendable products.

**A.** We will guarantee the quality of our equipment, normally there is no quality problem

1. If there is a deficiency, please email or call us on the reasons, it would be best to dealt with by email or telephone. Because some of the problems are caused by using the machine in a wrong way.
2. We suggest the client to take photos of the area where the problem occurs, we will pass them to our technician , our technician will give a satisfactory answer within 1-7 working days
3. If the problem comes from the spare part within the warranty, we could consider in changing the spare part to our client, which could be sent with his/her next order. If the next order is uncertain, we could pay freight cost to send the spare parts.
4. If the problem comes from the main device within the warranty, we could advice our client to post the device back to our company for repair by ordinary post, or we could re-send a new one to our client. If the cost of posting the problematic device back to BEST is more than the value of a new device, BEST would take it into consideration, according to the actual status; apply to re-send a completely new device back to our client.

**B.** We won't supply maintaining guarantee if troubles as below:

- a) Disassembling the machine by self;
- b) Fall it without caution;
- c) Lack of reasonable keeping and protect;
- d) Not operate according to the user manual.
- e) Expendable reasons

If changes of the design and specification of the machine, we will not notice you in addition. If any queries, please feel free to contact with the local marketing company.

All in one, we will try our best to serve you and also need your fully cooperation. Thanks.

<b>Product name :</b>			<b>Purchase date:</b>			<b>deadline:</b>		
<b>Seller:</b>			<b>Add.:</b>			<b>Tel.:</b>		
<b>Customer:</b>			<b>Add.:</b>			<b>Tel.:</b>		
<b>Repair date</b>			<b>Reason of the fault</b>			<b>Repair content</b>		
<b>Maintain list</b>								

